DELUXE 600
Upper Arm Blood Pressure Monitor**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT**
Keep this manual in a safe location for future reference**ENGLISH****1. IMPORTANT SAFETY NOTES****Safety and symbols**

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

READ THIS ENTIRE MANUAL, THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
	Indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	Indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
	Addresses practices not related to personal injury, such as product and/or property damage.
	Galvanically isolated applied part (F stands for "floating"); meets the requirements for leakage currents for type B
	Read the instructions before starting work and/or operating devices or machines
	The device is suitable for use with direct current only
	Note on important information
	Temperature limitation
	Humidity limitation
	Serial number
	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water
	Data transfer via Bluetooth® low energy technology

WARNING

- Do not use the blood pressure monitor on newborns or patients with pre-eclampsia. Consult your doctor before using the blood pressure monitor during pregnancy.
- This device may have difficulty determining the proper blood pressure for pregnant women and for users with irregular heartbeat, diabetes, poor circulation of blood, kidney problems, or for users who have suffered from a stroke.
- The pulse display is not suitable for checking the frequency of heart pacemakers or defibrillators.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Supervise children around the device to ensure they do not play with it.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- Make sure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access intravascular therapy, or an arteriovenous (AV) shunt.
- Do not use the device at the same time as other medical electrical devices (ME equipment). This could lead to a malfunction of the device and/or an inaccurate measurement.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions remove the cuff from the arm.

CAUTION

- Place the cuff on your upper arm only. Do not place the cuff on other parts of the body.
- Please note that when inflating, the functions of the limb in question may be impaired.
- Do not perform measurements more frequently than necessary. The resulting restriction of the blood flow may cause injury.

NOTICE:

- Do not use the device outside of the specified storage and operating conditions. This could lead to incorrect measurements.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.
- Using the blood pressure monitor outside your home environment or whilst on the move (e.g. whilst travelling in a car, ambulance or helicopter, or whilst undertaking physical activity such as playing sport) can influence the measurement accuracy and cause incorrect measurements.
- The blood pressure monitor is made from precision and electronic components. The accuracy of the measured values and service life of the device depend on its careful handling.

- Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.

- Do not drop the device.

- Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.

- Only use the cuff included with the delivery or original replacement cuffs. Otherwise incorrect measured values will be recorded.

- Bring the device to room temperature before taking measurements. If the measuring device has been stored near to the maximum or minimum storage and transport temperature and is brought into an environment with a temperature of 68 °F (20 °C), it is recommended that you wait 2 hours before using the measuring device.

- Never immerse or spill water or any other liquid onto the monitor or any components, otherwise liquid will enter it and cause damage.

- Never attempt to repair, open and/or disassemble the unit (including arm cuff and optional accessory) or adjust it yourself. This may damage the unit and impair the functions. If you need to have the unit repaired, please contact our customer service. Please see warranty for service contact. Before submitting any complaint, first check the batteries and replace them if necessary.

- Only use the device on people who have the specified upper arm measurement for the device.

- Avoid any mechanical restriction, compression or bending of the cuff line.

- To conserve the batteries, the blood pressure monitor switches off automatically if no buttons are pressed for 1 minute.

- Do not expose the device to static electricity. Always ensure that no static electricity is coming off of you before you operate the device.

Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.

- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.

For Canada only / Solo para Canada / Pour le Canada seulement:
Manufacturer / Fabricante / Fabricant : Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA
www.shop-beurer.com

Distributed by / Distribuido por / Distribue par : Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA
www.shop-beurer.com

3. The month flashes on the LCD Display. Set the month with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.
 4. The day flashes on the LCD Display. Set the day with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.
NOTE: If the hour format is set to 24h, the day/month display sequence is reversed.
 5. The hour flashes on the LCD Display. Set the hour with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.
 6. The minutes flash on the LCD Display. Set the minutes with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.
 7. The **Bluetooth®** icon flashes on the LCD Display. Activate ("On" flashes) or deactivate ("Off" flashes) **Bluetooth®** data transfer with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm. Note that operation with **Bluetooth®** activated will increase battery drain.

Bluetooth® Setup

In addition to displaying and saving measurements locally on the blood pressure monitor itself, you can also transfer measurements to your mobile device via **Bluetooth®**. To do so, you will need to install the "beurer HealthManager Pro" app. The app is freely available in either the Apple App Store or Google Play Store.

List of system requirements and compatible devices:

To transfer the measured values, proceed as follows:


- Activate **Bluetooth®** in the smartphone settings.
- Start the app.
- Select DELUXE 600 in the app and follow the instructions.

6. MEASURING BLOOD PRESSURE


The blood pressure monitor is suitable for home use and needs to be at room temperature when measuring. Measurements can be performed on either arm.

Positioning the Cuff:

1. Place the cuff on a bare upper arm. Ensure blood circulation is not restricted by tight clothing or other objects. The cuff must be placed on the upper arm so that the bottom edge is positioned 0.8 in - 1.2 above the elbow and over the artery, with the Air Tube in line with the center of the palm.
2. Fasten the cuff, making sure it is not too tight around your arm. You should be able to fit two fingers under the cuff after it is in place. The cuff is suitable for you if the index mark is within the OK range on the cuff when it is in place.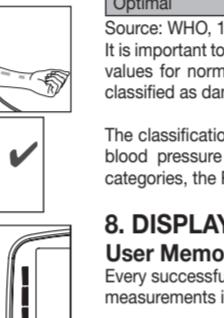

3. Insert the Air Tube Connector into the Air Tube Port.

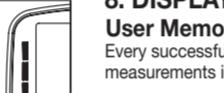Blood pressure can vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm. If the values between the two arms are significantly different, please consult your doctor to determine which arm should be used for the measurement.

Cuff Position Indicator

This device features a cuff position indicator to check the correct positioning of the cuff on your upper arm during measurement.

When the cuff is correctly positioned, the symbol will appear in the LCD Display during measurement and will also display along with measurement results.

If the cuff is positioned incorrectly, the measurement is cancelled and an error message appears in the LCD Display. If this happens, refer to the "Troubleshooting Guide" section for solutions.

Correct Posture:

- For consistency, always measure your blood pressure at the same time each day.

• Rest for at least five minutes before each measurement.

• Perform the measurement either sitting or lying down. Always make sure that the cuff is level with your heart.

• To perform a measurement, sit comfortably with your arms and back supported. Do not cross your legs and place your feet flat on the floor.

• Keep still during the measurement and do not talk.

• Relax your arm and the palm of your hand.

• When performing multiple measurements on an individual, wait at least 5 minutes between each measurement.

• Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.

• Repeat the measurement if you are unsure of the results.

• Measurements are for your information only – they are no substitute for a medical examination. Discuss the measurements with your doctor and never base any medical decisions on them (e.g. medicines and their administration).

The data has been deleted.

Clearing Memory:

- To delete the memory of the selected user, press the **MEMORY** button when the device is switched on.

• The average value of all measurements will appear on the display. Next to this, "AVG" and the **Bluetooth®** transfer symbol will flash, if **Bluetooth®** function is turned on.

• Keep the function buttons \wedge/\vee pressed simultaneously for 5 seconds.

• After successive deletion, "CL_0_" will appear in the display.

The data has been deleted.

Restoring the device to its factory settings

You can delete all of the data saved on the device, e.g. settings or saved measured values, by restoring the device to its factory settings.

Proceed as follows:

- With the device switched on, press the **MEMORY** button. You will then be in the most recently selected user memory. "AVG" will flash in the display.

- Now press the \wedge/\vee button again to switch to the display mode for the average morning value. will flash in the display.

- Hold down both function buttons \wedge/\vee at the same time for 15 seconds, until "CL_" appears in the display. Now all of the data saved on the device will be deleted and the device will be restored to its factory settings.

Performing a Measurement

1. There are 2 user memories available, each with 90 memory spaces, so that you can save the measured values of 2 different people separately. To select a desired user memory, press the **START/STOP** button ① and then press the **User Button** until the desired user memory icon appears in the LCD Display. Confirm by pressing the **START/STOP** button again; the device will then automatically begin measuring by inflating the cuff.

2. The measurement is taken while the cuff is being inflated. As soon as a pulse is detected, the symbol will flash in the rhythm of your pulse. After no pulse is detected anymore, the cuff air pressure will be released.

3. The measurements for systolic pressure, diastolic pressure and pulse will be displayed.

You can stop the measurement at any time by pressing the **START/STOP** button ①.

4. The measurement will be saved automatically.

5. will appear if the measurement was not performed properly. Refer to the "Troubleshooting Guide" section for details about specific error message numbers and then repeat the measurement.

6. If you have activated **Bluetooth®** data transfer, the data will be transferred. The blood pressure monitor displays the symbol for **Bluetooth®** transfer during transfer. To power down press the **START/STOP** button ①.

Note: The device switches off automatically after one minute if no button has been pressed.

Wait for at least five minutes before taking another measurement!

7. EVALUATING RESULTS**Irregular Heartbeat:**

This device can detect irregular heartbeat patterns during measurement and will indicate them by displaying the Irregular Heartbeat icon . This could be an indicator for arrhythmia, a condition where the heart rhythm is abnormal. These symptoms (omitted or premature heartbeats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use of stimulants, stress, or lack of sleep. This function is not designed for diagnosing or treating an arrhythmic disorder. Only your doctor can determine if you do have arrhythmia.

If the battery replacement symbol is permanently displayed, you can no longer perform any measurements and must replace all batteries. Once the batteries have been removed from the device, the date and time must be set again. Any saved measurements are retained.

Repeat the measurement if the Irregular Heartbeat icon displays after a measurement. Rest for at least five minutes between measurements and do not talk or move during measurement. If the icon appears often, contact your doctor, since self-diagnosis and treatment based solely on the test results may be dangerous. It is vital to follow your doctor's instructions.

Resting indicator (using HSD diagnostics)

One of the most common errors made when taking a blood pressure measurement is not ensuring that the user's circulatory system is sufficiently at rest when taking the measurement.

In this case, the measured systolic and diastolic blood pressure values do not represent the blood pressure at rest. However, it is this blood pressure at rest that should be used to assess the measured values.

This blood pressure monitor uses integrated haemodynamic stability diagnostics (HSD) to measure the user's haemodynamic stability when taking the blood pressure measurement. This enables it to indicate whether the blood pressure was taken when the user's circulatory system was sufficiently at rest.

Before first use and after every battery change:

When you insert batteries into the device, you will automatically arrive at the relevant menu.

If batteries have already been inserted:

With the device switched-on press and hold the **Bluetooth®** Button for 3 seconds.

1. The time format will flash. Select an hour format (12- or 24-hour display) with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.

2. The year flashes on the LCD Display. Set the year with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.

3. The month flashes on the LCD Display. Set the month with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.

4. The day flashes on the LCD Display. Set the day with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.

NOTE: If the hour format is set to 24h, the day/month display sequence is reversed.

5. The hour flashes on the LCD Display. Set the hour with the \wedge/\vee Buttons and press the **Bluetooth®** Button to confirm.

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The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer North America LP is under license. Other trademarks and trade names are those of their respective owners. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple LLC, registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC. Android is a trademark of Google LLC.