For Canada only / Solo para Canada /  
Pour le Canada seulement:  
Manufacturer / Fabricante / Fabricant :  
Beurer North America LP  
1 Oakwood Boulevard, Suite 255  
Hollywood, FL 33020, USA  
www.shop-beurer.com

Distributed by / Distribuido por / Distribue par :  
Beurer GmbH, Soflinger Str. 218,  
89077 Ulm, Germany

## ENGLISH

## 1. IMPORTANT SAFETY NOTES

## Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

**READ THIS ENTIRE MANUAL, THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
	Indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	Indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
	Addresses practices not related to personal injury, such as product and/or property damage.
	Galvanically isolated applied part (F stands for "floating"); meets the requirements for leakage currents for type B
	Read the instructions before starting work and/or operating devices or machines
	The device is suitable for use with direct current only
	Note on important information
	Temperature limitation
	Humidity limitation
	Serial number
	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water

**WARNING**


- Do not use the blood pressure monitor on newborns or patients with pre-eclampsia. Consult your doctor before using the blood pressure monitor during pregnancy.
- This device may have difficulty determining the proper blood pressure for pregnant women and for users with irregular heartbeat, diabetes, poor circulation of blood, kidney problems, or for users who have suffered from a stroke.
- The pulse display is not suitable for checking the frequency of heart pacemakers or defibrillators.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Supervise children around the device to ensure they do not play with it.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- Make sure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access intravascular therapy, or an arteriovenous (AV) shunt.
- Do not use the device at the same time as other medical electrical devices (ME equipment). This could lead to a malfunction of the device and/or an inaccurate measurement.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions remove the cuff from the arm.

**CAUTION**

- Place the cuff on your upper arm only. Do not place the cuff on other parts of the body.
- Please note that when inflating, the functions of the limb in question may be impaired.
- Do not perform measurements more frequently than necessary. The resulting restriction of the blood flow may cause injury.

**NOTICE:**

- Do not use the device outside of the specified storage and operating conditions. This could lead to incorrect measurements.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.
- Using the blood pressure monitor outside your home environment or whilst on the move (e.g. whilst travelling in a car, ambulance or helicopter, or whilst undertaking physical activity such as playing sport) can influence the measurement accuracy and cause incorrect measurements.
- The blood pressure monitor is made from precision and electronic components. The accuracy of the measured values and service life of the device depend on its careful handling:
  - Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
  - Do not drop the device.
  - Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.
  - Only use the cuff included with the delivery or original replacement cuffs. Otherwise incorrect measured values will be recorded.
  - Bring the device to room temperature before taking measurements. If the measuring device has been stored near to the maximum or minimum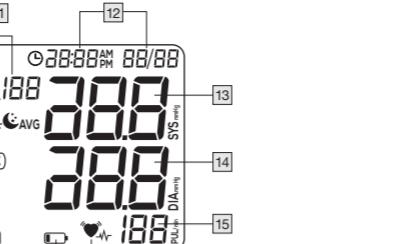


11. Memory space number  
12. Time and Date  
13. Systolic Pressure  
14. Diastolic Pressure  
15. Pulse value determined  
16. Irregular Heartbeat Indicator  
17. Low Battery Icon  
18. User memory  
19. Risk Indicator  
20. Cuff position indicator

21. Memory display: average value

AVG, morning ☀, evening ☁

## 5. PREPARING FOR MEASUREMENT

## Inserting/changing batteries: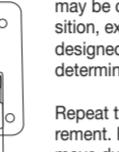

1. Remove the Battery Compartment Cover from the rear of the device.
2. Insert four "AAA" alkaline batteries (included) according to the polarity markings inside the compartment. **NOTE:** Do not use rechargeable batteries.
3. Reattach the Battery Compartment Cover.
4. All LCD Display elements will briefly appear and "12h" will flash. Set the date and time as described below. Any saved measurements will be retained.

If the battery replacement symbol is permanently displayed, you can no longer perform any measurements and must replace all batteries. Once the batteries have been removed from the device, the date and time must be set again. Any saved measurements are retained.

## Setting the hour format, date, and time:

If date and time are not set, stored measurements will not have a valid timestamp associated with them.

The settings menu can be accessed in two different ways:

## Before first use and after every battery change:

When you insert batteries into the device, you will automatically arrive at the relevant menu.

## If batteries have already been inserted:

With the device switched-on press and hold the **MEMORY** Button for 3 seconds. 1. The time format will flash. Select an hour format (12- or 24-hour display) with the **>** button and press the **MEMORY** Button to confirm.

2. The year flashes on the LCD Display. Set the year with the **>** button and press the **MEMORY** Button to confirm.

3. The month flashes on the LCD Display. Set the month with the **>** button and press the **MEMORY** Button to confirm.

4. The day flashes on the LCD Display. Set the day with the **>** button and press the **MEMORY** Button to confirm.

NOTE: If the hour format is set to 24h, the day/month display sequence is reversed.

5. The hour flashes on the LCD Display. Set the hour with the **>** button and press the **MEMORY** Button to confirm.


6. The minutes flash on the LCD Display. Set the minutes with the **>** button and press the **MEMORY** Button to confirm.

## 6. MEASURING BLOOD PRESSURE

The blood pressure monitor is suitable for home use and needs to be at room temperature when measuring. Measurements can be performed on either arm.

## Positioning the Cuff:

1. Place the cuff on a bare upper arm. Ensure blood circulation is not restricted by tight clothing or other objects. The cuff must be placed on the upper arm so that the bottom edge is positioned 0.8 in - 1.2 in above the elbow and over the artery, with the Air Tube in line with the center of the palm.

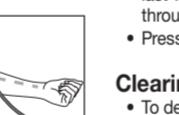2. Fasten the cuff, making sure it is not too tight around your arm. You should be able to fit two fingers under the cuff after it is in place. The cuff is suitable for you if the index mark is within the OK range on the cuff when it is in place.

3. Insert the Air Tube Connector into the Air Tube Port.

Blood pressure can vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm. If the values between the two arms are significantly different, please consult your doctor to determine which arm should be used for the measurement.

## Cuff Position Indicator

This device features a cuff position indicator to check the correct positioning of the cuff on your upper arm during measurement.

When the cuff is correctly positioned, the symbol will appear in the LCD Display during measurement and will also display along with measurement results. If the cuff is positioned incorrectly, the measurement is cancelled and an error message appears in the LCD Display. If this happens, refer to the "Troubleshooting Guide" section for solutions.

## Correct Posture:

- Rest for at least five minutes before each measurement.
- Perform the measurement either sitting or lying down. Always make sure that the cuff is level with your heart.
- To perform a measurement, sit comfortably with your arms and back supported. Do not cross your legs and place your feet flat on the floor.
- Keep still during the measurement and do not talk.
- Relax your arm and the palm of your hand.

## LCD Display Information

## 1. Universal cuff

## 2. Air Tube

## 3. Air Tube Connector

## 4. Connection for Air tube connector

## 5. Risk Indicator

## 6. LCD Display

7. **MEMORY** Button8. **START/STOP** Button ①9. **>** Button

## 10. Battery Compartment Cover

## Performing a Measurement

1. With the cuff attached properly and in a correct posture, start the device using the **START/STOP** Button ①. Press the **START/STOP** Button ① again; the device will then automatically begin measuring by inflating the cuff.
2. The measurement is taken while the cuff is being inflated. As soon as a pulse has been detected, the pulse symbol will flash in the rhythm of your pulse. After no pulse is detected anymore, the cuff air pressure will be released.
3. The measurements for systolic pressure, diastolic pressure and pulse will be displayed. You can stop the measurement at any time by pressing the **START/STOP** button ①.
4. The measurement will be saved automatically.
5. "**Er\_**" will appear if the measurement was not performed properly. Refer to the "Troubleshooting Guide" section for details about specific error message numbers and then repeat the measurement.

**Note:** The device switches off automatically after 60 seconds if no button has been pressed.

Wait for at least five minutes before taking another measurement!

## 7. EVALUATING RESULTS

## Irregular Heartbeat:

This device can detect irregular heartbeat patterns during measurement and will indicate them by displaying the Irregular Heartbeat icon . This could be an indicator for arrhythmia, a condition where the heart rhythm is abnormal. These symptoms (omitted or premature heartbeats, slow or excessively fast heart rate) may be caused, among other things, by heart disease, age, physical predisposition, excessive use of stimulants, stress, or lack of sleep. This function is not designed for diagnosing or treating an arrhythmic disorder. Only your doctor can determine if you do have arrhythmia.

Repeat the measurement if the Irregular Heartbeat icon displays after a measurement. Rest for at least five minutes between measurements and do not talk or move during measurement. If the icon appears often, contact your doctor, since self-diagnosis and treatment based solely on the test results may be dangerous. It is vital to follow your doctor's instructions.

## Risk Classification

Measurements can be classified and assessed by the table below. However, these standard values are only a general guideline, since individual blood pressure varies in different people and different age groups, etc.

Hypertension Category	Systolic (in mmHg)	Diastolic (in mmHg)	LED Risk Indicator
Grade 3: Severe hypertension	≥180	≥110	Red
Grade 2: Moderate hypertension	160–179	100–109	Orange
Grade 1: Mild hypertension	140–159	90–99	Yellow
High Normal	130–139	85–89	Green
Normal	120–129	80–84	Green
Optimal	<120	<80	Green

Source: WHO, 1999

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The classification on the display and the scale on the unit show which category the recorded blood pressure values fall into. If the systolic and diastolic values fall into two different categories, the Risk Classification Indicator on the device always shows the higher category.

## 8. DISPLAYING AND DELETING MEASUREMENTS

## User Memory:

Every successful measurement is stored along with its date and time. If there are more than 60 measurements, the oldest measurement is deleted.

## Average Values:

- With the device switched-on press and hold the **MEMORY** button for 3 seconds. The average value of all measurements will appear on the display.
- Press the **>** button to display the average value of the last 7 days for the morning measurement. (Morning: 5:00 am-9:00 am, display ). Press the **>** button again to display the average value of the last 7 days for the evening measurement. (Evening: 6 pm-8 pm, display ). By pressing the **>** button again, the last individual measurement will be displayed with the date and time. Switch through all the saved measurements by pressing the **>** button again.
- Press the **START/STOP** button ① for 2 seconds to switch off the device.

## Clearing Memory:

- To delete the memory, press the **MEMORY** button when the device is switched on.
- The average value of all measurements will appear on the display.
- Keep the **>** button and **MEMORY** button pressed simultaneously for 5 seconds.
- After successful deletion, "**CL 01**" will appear in the display. The data has been deleted.

## 9. CARE AND MAINTENANCE

- Clean the device and cuff with a slightly moistened cloth.
- Do not use detergents or solvents.
- Never immerse the device in water or other liquids.
- When storing the device, make sure no heavy objects are placed on top of it. Remove the batteries. Ensure the Cuff line does not have any sharp kinks.

## Disposal:

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

## 10. TROUBLESHOOTING GUIDE

In case of errors, "**Er\_**" will appear in the LCD Display, where **\_** is a number 1-6. Error messages may appear if:

Trouble shooting	Problem	Solution
"Er1"	The pulse cannot be measured correctly.	Repeat the measurement process. Be sure not to move or talk.
"Er2"	You have been talking or moving during the measurement process.	</

Table 3

Guidance and manufacturer's declaration - electromagnetic Immunity

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum Power (W)	Distance (m)	Test Level (V/m)	Compliance level (V/m)	
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	385	380-390	TETRA 400	Pulse modulation 18 Hz	1,8	0.3	27	27
	450	430-470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	2	0.3	28	28
	710	704-787	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9	9
	745			Pulse modulation 18 Hz	2	0.3	28	28
	780			Pulse modulation 217 Hz	0.2	0.3	9	9
	810	900-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28	28
	870			Pulse modulation 217 Hz	0.2	0.3	28	28
	930			Pulse modulation 217 Hz	0.2	0.3	28	28
	1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28	28
	1845			Pulse modulation 217 Hz	2	0.3	28	28
	1970			Pulse modulation 217 Hz	2	0.3	28	28
	2450	2400-2570	Bluetooth®, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28	28
	5240	5100-5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9	9
	5785	Pulse modulation 217 Hz	0.2	0.3	9	9		

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