**READ THIS MANUAL COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT**

Keep this manual in a safe location for future reference

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 **Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.**

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## Contents

1. IMPORTANT SAFETY NOTES ..... 2	9. Taking Measurements ..... 7
2. Intended use..... 4	10. Evaluating Results ..... 7
3. Getting to Know your Scale ..... 4	11. Other functions ..... 9
4. Package Contents ..... 4	12. Care, Maintenance and Disposal ..... 10
5. Parts and Controls ..... 5	13. Troubleshooting Guide ..... 10
6. Set up ..... 5	14. Technical Specifications ..... 11
7. Use with App ..... 6	15. FCC Compliance Information..... 12
8. Use without app ..... 6	16. Warranty ..... 12

## SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

## 1. IMPORTANT SAFETY NOTES

### Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

**READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.**

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
	<b>WARNING</b> Indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	<b>CAUTION</b> Indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
<b>NOTICE</b>	Addresses practices not related to personal injury, such as product and/or property damage.

	Follow instructions for use	<b>Storage</b> 	Permissible storage temperature and humidity
	Type BF applied part	<b>Operating</b> 	Permissible operating temperature and humidity
	Do not use the device with a pacemaker	<b>IP21</b>	Protected against access to hazardous parts with a finger and against vertically falling water drops

## **WARNING**

- The scale must not be used by people with medical implants (e.g. pacemakers), as this may affect their functionality.
- Do not use during pregnancy.
- To prevent falls, do not step onto the scale with wet feet or if the surface of the scale is damp.
- Do not stand on the outer edge of the scale, otherwise it may tip.
- Keep children away from packaging materials (risk of suffocation).

## **CAUTION**

- This device is not intended to diagnose or treat any medical condition.
- This product is intended solely for household use and not for medical or commercial purposes.
- Not for use by children under 10 years of age.
- Consult your physician or healthcare provider before beginning a weight reduction or exercise program.

## **NOTICE**

- The device should be cleaned from time to time. Do not use any abrasive cleaning products.
- Measurement variations are normal, as this scale is not calibrated for use in a professional medical environment.
- Do not place any objects on the scale when not in use.
- This product is intended solely for household use and not for medical or commercial purposes.
- Protect the device from impacts, moisture, dust, chemicals, large temperature fluctuations, and nearby sources of heat (ovens, heaters).
- Make sure that no liquids come into contact with the scale. Never submerge the scale in water. Never rinse it in running water.
- Do not expose the scale to high temperatures or strong electromagnetic fields.

## **Battery Handling Safety Precautions**

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.
- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions).
- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.

## **Notes on electromagnetic compatibility**

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.

- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

## **2. INTENDED USE**

This digital Body Analysis Scale is intended for measuring body weight and providing an analysis of your personal fitness data. It is intended for household use only. Measurement of body weight, body fat, body water, muscle ratio, bone mass, AMR/BMR and BMI.

## **3. GETTING TO KNOW YOUR SCALE**

This scale works on the principle of Bioelectrical Impedance Analysis (BIA). This involves a quick calculation of body content using a harmless electric current that is not even noticeable.

When measurement of the electrical resistance (impedance) is considered alongside other data (age, height, gender, and activity level), body fat percentage and other variables in the body can be estimated. Muscle tissue and water conduct electricity well and therefore have a lower resistance.

In contrast, bones and fat tissue have low conductivity as fat cells and bones only minimally conduct current due to very high levels of resistance.

Calculated values only approximate the actual values for your body. Only a doctor can precisely establish body fat, body water, muscle percentage and bone structure using medical methods.

### **Tips**

- If possible, always weigh yourself at the same time every day (ideally in the morning), after having used the restroom, on an empty stomach and with as little clothing as possible, in order to obtain comparable, consistent results.
- Body fat percentage can only be calculated when you are barefoot; the soles of the feet may be lightly dampened where appropriate. Results may be unsatisfactory if the soles of your feet are completely dry or feature areas of rough skin; this reduces conductivity.
- Stand up straight and still while weighing.
- Wait a few hours after significant physical exertion.
- Wait about 15 minutes after waking so that your body's water content can normalize.
- Remember that only long-term trends are important. Brief deviations in weight within a few days are normally caused by changes in body water content.

### **Limitations**

When calculating body fat and additional values, deviating and unexpected results may occur in the case of:

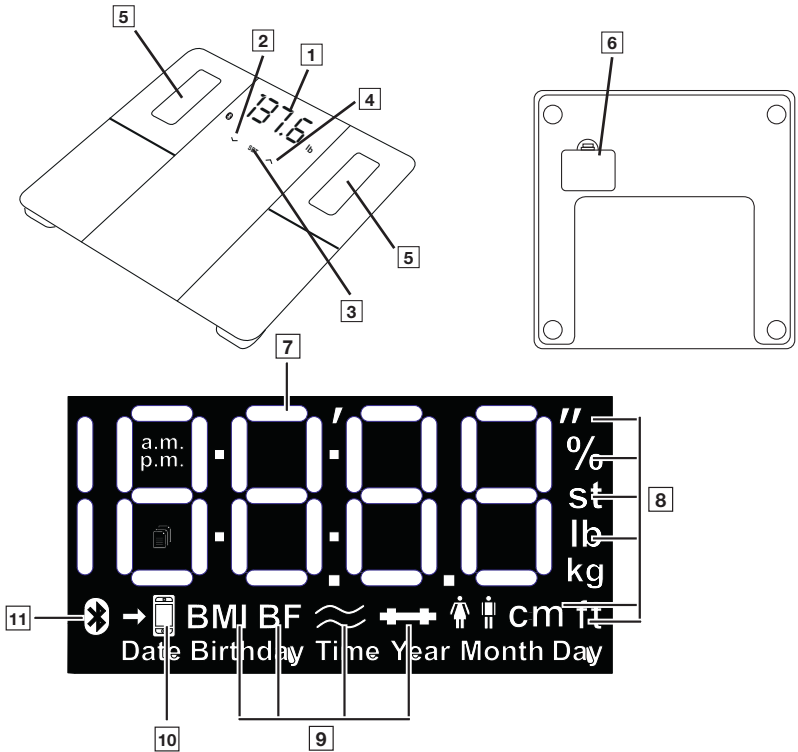
- Children under 10 years of age
- Athletes and bodybuilders
- Pregnant women
- Persons with a fever, persons undergoing dialysis, persons demonstrating edema symptoms, and persons suffering from osteoporosis
- Persons taking cardiovascular medicine (affecting the heart and vascular system)
- Persons taking vasodilating or vasoconstricting medication
- Persons with significant anatomical deviations in their legs compared with their overall body height (leg length significantly shorter or longer).

## **4. PACKAGE CONTENTS**

1 x BF 500 Body Analysis Scale  
3 x 1.5 V AA batteries

- 1 x Instruction manual
- 1 x Quick start guide

## 5. PARTS AND CONTROLS

- |  |                                     |
|--|-------------------------------------|
| <b>1</b> Display   | <b>7</b> Measurements               |
| <b>2</b> √-button (reduces setting value)  | <b>8</b> Measurement unit           |
| <b>3</b> SET button (to confirm)   | <b>9</b> Body data                  |
| <b>4</b> ^-button (increases setting value or for user selection when assigning measured values) | <b>10</b> Measurement data transfer |
| <b>5</b> Electrodes  | <b>11</b> Bluetooth® symbol         |
| <b>6</b> Battery compartment   |                                     |


## 6. SET UP
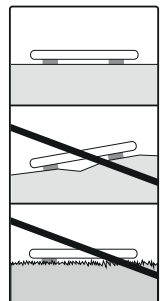
### Inserting the Batteries

Remove the Battery Compartment Cover and insert three AA-size alkaline batteries (included) according to the indicated polarity. Replace Battery Compartment Cover.

### Positioning the scale

Place the scale on a hard, flat surface with no carpet or rugs; a hard surface is required for accurate measurements.

## 7. USE WITH APP

### Downloading the App

The scale has 30 memory spaces for 8 different scale users, enabling you to save the various members of your family, for example. You can use the “beurer HealthManager Pro” app to conveniently set up the individual app users and their corresponding scale user via your smartphone. To do so, proceed as follows:

- Download the free “beurer HealthManager Pro” app from the Apple App Store or Google Play.

Click here for the “beurer HealthManager Pro” app

List of system requirements and compatible devices

- Activate *Bluetooth*® in your smartphone’s settings.
- Start the app.
- Select BF 500 in the app and follow the instructions.

User data	Set values
Initials/abbreviation of name	Max. 3 letters or numbers
Height	3 ft 3.5 in to 7 ft 2.5 in (100 to 220 cm)
Age	10 to 100 years
Gender	Male (♂), female (♀)
Activity level	1 to 5

To select the proper Activity Level, consult the table below.

Activity level	Physical activity
1	None.
2	Low: little and light physical exertion (e.g. going for walks, light gardening, simple gymnastics).
3	Medium: physical exertion, at least 2 to 4 times a week for 30 minutes each.
4	High: physical exertion, at least 4 to 6 times a week for 30 minutes each.
5	Very high: intensive physical exertion, intensive training or hard physical work for at least 1 hour every day.

### Note on multi-user usage:

Create a separate app user for each scale user (in the same or on another smartphone/tablet with “beurer HealthManager Pro” app).

If the same scale is used by several app users, the device must be added in the app for each of them during setup. A new user is automatically created on the scale (e.g. U:2).

## 8. USE WITHOUT APP

Alternatively, you can create a scale user directly on the scale without using the app. Do not create a scale user directly on the scale if you have already created that scale user using the app. This process is not required if the scale has already been connected to the app.

- After inserting the batteries, press the **SET button**.
- “Unit” appears in display, select with  $\wedge/\vee$  button and confirm with **SET button**.

**NOTE:** The scale is initially set to weigh and measure in “lb” and “in”.

- “USEr” appears in display. Use the  $\wedge$  button to select the scale user you need (you can select from U: 1 to 8). Confirm the selection with **SET button**.
- (♂) flashes on the display. Use the  $\vee$  button to set the gender. Confirm your entry with the **SET button**.
- “5,7” flashes on the display. Use the  $\wedge$  button to set the height. Confirm your entry with the **SET button**.

6. The year, then the month, then the day flash on the display. Use the **^** button to set the year, month and day of birth. Confirm your entry with the **SET button**.
7. “3” flashes on the display. Use the **^/√** button to set your activity level based on how much you exercise (considered over the long term). (1 = none, 2 = low, 3 = medium, 4 = high, 5 = very high/athletic).
8. Confirm your entry with the **SET button**.

**NOTE:** If you used the scale the first time without the app, you can add your existing scale user memory to the “beurer HealthManager Pro” app. This requires you to enter your 4-digit user PIN. This process is described in the app.

To set automatic user recognition, the first measurement must be assigned to your personal user data; see below for instructions.

## 9. TAKING MEASUREMENTS

**NOTE:** Do not let your feet, legs, calves or thighs touch each other. Otherwise, the measurement cannot be taken correctly. The measurement will also be incorrect if taken while you are wearing socks.

1. Step onto the scale with bare feet; stand still with your weight distributed evenly on both sets of sensors.
2. Your body weight appears on the display. The scale user detected (e.g. “U: 1” for scale user 1) should then appear automatically.

**NOTE:** If the scale has detected several possible scale users, use the **^** button to select the correct one and wait 3 seconds. Step off the scale.

3. The following health values will then appear one after the other.

<b>BMI</b>	Body mass index
<b>BF</b>	Body fat in %
	Body water content in %
	Muscle content in %

**NOTE:** If you are wearing shoes when taking the measurement, only your weight will be shown (the weight is automatically saved when the scale detects the scale user).

4. If a *Bluetooth*<sup>®</sup> connection has been established and the user measurements have been successfully taken (e.g. for U: 1), the values will be transferred to the “beurer HealthManager Pro” app. This is indicated by the *Bluetooth*<sup>®</sup> symbol. If the data cannot be transferred, up to 30 values will be stored per user. They will be transferred as soon as a connection has been established.

**NOTE:** If the automatic scale user detection function is not working, press the **^** button until your scale user is displayed, and wait 3 seconds. Then take the diagnostic measurement.

5. The scale will shut off by itself after a few moments of inactivity.

## 10. EVALUATING RESULTS

### Body fat percentage

The following body fat percentages are a guideline only; for more information, ask your doctor.

#### Male

Age	Low	Normal	Moderate	High
10–14	<11%	11–16%	16.1–21%	>21.1%
15–19	<12%	12–17%	17.1–22%	>22.1%
20–29	<13%	13–18%	18.1–23%	>23.1%
30–39	<14%	14–19%	19.1–24%	>24.1%
40–49	<15%	15–20%	20.1–25%	>25.1%

#### Female

Age	Low	Normal	Moderate	High
10–14	<16%	16–21%	21.1–26%	>26.1%
15–19	<17%	17–22%	22.1–27%	>27.1%
20–29	<18%	18–23%	23.1–28%	>28.1%
30–39	<19%	19–24%	24.1–29%	>29.1%
40–49	<20%	20–25%	25.1–30%	>30.1%

### Male

Age	Low	Normal	Moderate	High
50–59	<16%	16–21%	21.1–26%	>26.1%
60–69	<17%	17–22%	22.1–27%	>27.1%
70–100	<18%	18–23%	23.1–28%	>28.1%

### Female

Age	Low	Normal	Moderate	High
50–59	<21%	21–26%	26.1–31%	>31.1%
60–69	<22%	22–27%	27.1–32%	>32.1%
70–100	<23%	23–28%	28.1–33%	>33.1%

Athletes will usually have lower values. Depending on the sport, training intensity, and your physical constitution, results may still be below the stated ranges. Please note that there may be health risks in the case of extremely low values.

### Body Water Content

Body water content in percent is normally within the following ranges:

#### Male

Age	Poor	Good	Very good
10–100	<50%	50–65%	>65%

#### Female

Age	Poor	Good	Very good
10–100	<45%	45–60%	>60%

Body fat contains relatively little water; therefore, body water content may be below the standard values in persons with a high body fat percentage. On the other hand, some athletes may exceed the standard values due to having low levels of fat and a high muscle percentage.

The body water calculation performed using this scale is not suitable for drawing medical conclusions, such as in the case of age-related water retention. Consult your doctor where necessary. A high body water content is generally desirable.

### Muscle Percentage

The muscle percentage is normally within the following ranges:

#### Male

Age	Low	Normal	High
10–14	<44%	44–57%	>57%
15–19	<43%	43–56%	>56%
20–29	<42%	42–54%	>54%
30–39	<41%	41–52%	>52%
40–49	<40%	40–50%	>50%
50–59	<39%	39–48%	>48%
60–69	<38%	38–47%	>47%
70–100	<37%	37–46%	>46%

#### Female

Age	Low	Normal	High
10–14	<36%	36–43%	>43%
15–19	<35%	35–41%	>41%
20–29	<34%	34–39%	>39%
30–39	<33%	33–38%	>38%
40–49	<31%	31–36%	>36%
50–59	<29%	29–34%	>34%
60–69	<28%	28–33%	>33%
70–100	<27%	27–32%	>32%

### Other values that can be displayed in the app

#### Bone mass

Like the rest of our body, bones are subject to natural growth, shrinking, and aging processes. Bone mass increases rapidly during childhood and reaches its peak between the ages of 30 and 40. As we age, our bone mass begins to decrease. You can combat this reduction to an extent with the help of a healthy diet (particularly calcium and vitamin D) and regular physical exercise. The stability of the skeletal system may be increased through increased muscle mass. Note that this scale does not measure calcium content of bones; instead, it calculates the weight of all the components that make up the bones (organic matter, inorganic matter, and water). It is very difficult to affect your bone mass, although it does fluctuate within the scope of influencing factors (weight, height, age, gender). There are no recognized guidelines or recommendations.

NOTE: Do not confuse bone mass with bone density. Bone density can only be determined by a medical examination, so you cannot draw conclusions on changes to bones and bone hardness (e.g. osteoporosis) using this scale.

## **BMR**

The basal metabolic rate (BMR) is the amount of energy required by the body at complete rest in order to maintain its basic functions. This value primarily depends on weight, height and age. It is displayed in kcal/day and calculated using the scientifically recognized Harris-Benedict equation. This is the amount of energy that is required by your body under all circumstances and must be resupplied to the body in the form of food. If your energy intake is below this level long term, it can affect your health.

## **AMR**

The active metabolic rate (AMR) is the amount of energy that an active body consumes each day. An individual's energy requirement increases when the level of physical activity increases; the scale calculates this by means of the activity level (1– 5) that has been entered.

To maintain the current weight, the energy that the body uses must be replaced in the form of food and drink. If less energy is taken in than is used over a long period of time, the body takes the difference from its fat stores, and weight loss results. However, if the amount of energy taken in exceeds the calculated active metabolic rate (AMR) for a longer period, the body cannot burn off the excess energy. The excess is stored in the body as fat, leading to weight gain.

## **Interpreting Results**

**NOTE:** Only long-term trends are important. Brief deviations in weight within a few days are normally caused by loss of fluids.

Interpretation of these results is based on changes in total body weight, percentage of body fat, body water and muscle content, as well as the length of time over which these changes occur. Rapid changes within the scope of a few days are considered separately from medium-term changes (in the scope of weeks) and long-term changes (months). As a basic rule, short-term changes in weight are almost entirely changes in water content, whereas medium-term and long-term changes may indicate changes in fat percentage and muscle percentage.

- If you experience short-term weight loss but your body fat percentage increases or stays the same, you have only lost water, such as after a training session, visit to the sauna, or a diet aimed only at fast weight loss.
- On the other hand, if your weight increases in the medium term and your body fat percentage drops or stays the same, you may have increased your muscle mass.
- If your weight and body fat percentage both fall at the same time, then your diet is working – you are losing fat mass.
- Ideally, you should support your diet with physical activity, fitness or strength training. This enables you to increase your muscle percentage in the medium term.
- Body fat, body water, and muscle percentages should not be totaled, since muscle tissue also contains components made of body water.

# **11. OTHER FUNCTIONS**

## **Reviewing Saved Measurements**

To display your last diagnostic measurement, briefly press on the scale until “0.0” and the user symbol appear in the display, press  $\wedge$  button repeatedly until your scale user is displayed and wait 3 seconds. Then press the  $\vee$  button to display the most recent diagnostic measurement. To display further measurements, press the  $\vee$  button again. Up to 30 measurements can be shown on the display.

## **Deleting scale users**

You can delete individual scale users in the “beurer HealthManager Pro” app.

Alternatively, you can also delete scale users directly on the scale. To do so, proceed as follows:

1. Briefly press on the scale until “0.0” appears in the display.
2. Press the **SET** button.

3. “Unit” appears in the display, confirm with **SET button**.
4. “User” appears in display, select your user with **^** button and confirm with **SET button**.
5. Press the **SET button** repeatedly until “End” appears in the display. Press **^** button within 3 seconds, “dEL” will appear in the display. Confirm with the **SET button**, the user is deleted.

## 12. CARE, MAINTENANCE AND DISPOSAL

### Replacing Batteries

This scale is equipped with a low battery indicator. If the batteries are weak, “Lo” will appear in the LCD Display and the scale will shut off. Replace the batteries with three new AA-size alkaline batteries.

### Cleaning

Clean the scale periodically using a damp cloth and, if necessary, a small amount of mild detergent.

### Disposal

Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

## 13. TROUBLESHOOTING GUIDE

If the scale encounters an error during measurement, the following is displayed:

Display	Cause	Solution
<b>Discrepancy in weight</b>	Incorrect zero point.	Briefly press on the scale until the current weight is displayed, then remove your foot so that “0.0” flashes.
	Scale is not on a flat, stable surface.	Place the scale on an even, hard surface.
<b>PAIR</b>	<i>Bluetooth</i> <sup>®</sup> needs to be paired.	Start/allow pairing/bonding on your smartphone (note any messages in the status bar/notification bar), and confirm the 6-digit pairing code using the <b>SET button</b> on the device and confirm on the smartphone.
<b>“USE APP”</b>	Connect the scale using the app.	Do not connect it via the <i>Bluetooth</i> <sup>®</sup> settings on your smartphone. Only connect it via the app.
<b>U: -</b>	The scale was unable to assign the measurement to a scale user.	If the automatic scale user detection function is not working, briefly press on the scale, press the <b>button ^</b> repeatedly until your scale user is displayed, and wait 3 seconds. Weigh yourself again. The user assignment limit is +/-3 kg and can be changed in the app.
<b>BF-Err</b>	The body fat percentage is outside the measurable range (less than 3% or greater than 70%).	Check the user data or change the activity level.
<b>BF--</b>	Diagnostic measurement cannot be taken (e.g. measurement attempted while wearing shoes).	Repeat the measurement barefoot. Otherwise, only weight values will be displayed and transferred.

Display	Cause	Solution
<b>oLd</b>	The maximum weight capacity of 396 lb (180 kg/ 28 st) has been exceeded.	Loads must not exceed 396 lb (180 kg/ 28 st).
<b>Lo</b>	The batteries in the scale are empty.	Replace the batteries.
<b>No Bluetooth® connection  Symbol is not displayed</b>	Device is outside the <i>Bluetooth®</i> range.	Maximum unobstructed range is about 30 feet (~10 m). Walls and ceilings reduce range. Radio frequency waves may cause interference; do not place the scale near devices such as wireless routers or microwaves.
	No <i>Bluetooth®</i> connection to the app.	<ol style="list-style-type: none"> <li>1. Switch on the scale to establish the connection faster.</li> <li>2. Switch <i>Bluetooth®</i> on your smartphone off and on again. Ensure that the scale is not being connected via the <i>Bluetooth®</i> settings of the operating system. If necessary, remove it from the device list in the <i>Bluetooth®</i> settings and then use the “beurer HealthManager Pro” app.</li> <li>3. Completely close the app (including in the background). Switch your smartphone off and on again.</li> <li>4. Briefly remove the batteries from the scale and reinsert them. Launch the app.</li> <li>5. When connecting the scale, check the status bar on the smartphone to start pairing/bonding.</li> </ol>
<b>Measurements are not sent Measurement data transfer symbol is not displayed </b>	No measurement for the scale user or no <i>Bluetooth®</i> connection.	The user memory (e.g. U: 1) which appears on the scale display must be assigned to the scale in your app settings. Only these measurements can be transferred. Therefore, check your user on the scale when taking a measurement and in your app settings.

## 14. TECHNICAL SPECIFICATIONS

Type:	BF 500
Dimensions:	11.8 in x 11.8 in x 1.1 in (30 cm x 30 cm x 2.8 cm)
Weight:	4 lb (1.7 kg)
Measurement range:	396 lb/ 180 kg/ 28 st
Scale graduation d:	The display can be read in increments of 0.2 lb (0.1 kg)
Repetition accuracy:	The measuring tolerance for repeated measurements is +/- 0.8 lb (0.4 kg)
Absolute precision:	In comparison with a calibrated weight, the measured value is +/- 0.5% + 0.22 lb (0.1 kg). E.g. at 176 lb (80 kg) this corresponds to +/- 1.1 lb (0.5 kg)
Operating Conditions	41 °F to 95 °F (5 °C to 35 °C), 20-95 % RH
Storage Conditions	-4 °F to 140 °F (-20 °C to 60 °C), 20-95 % RH
Data transfer via <i>Bluetooth®</i> low energy technology	The product uses <i>Bluetooth®</i> low energy technology, frequency band 2402 - 2480 MHz, max. +3.1 dBm transmission power, compatible with <i>Bluetooth®</i> ≥ 4.0 smartphones/tablets

System requirements and list of compatible devices for the “beurer HealthManager Pro” app

## 15. FCC COMPLIANCE INFORMATION

Body Analysis Scale BF 500

Responsible Party – U.S. Contact Information

Beurer North America LP

1 Oakwood Boulevard, Suite 255

Hollywood, FL 33020

United States

1-800-536-0366

info@beurer.com

### FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

## 16. WARRANTY

Limited Lifetime Warranty For Original Purchaser

Your Beurer Body Analysis Scale, model BF 500, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners. We will, at our option, repair or replace the Beurer Body Analysis Scale, model BF 500, without additional charge, for any part or parts covered by these written warranties. No refunds will be given.

Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at [info@beurer.com](mailto:info@beurer.com) to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer.

This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

**IN NO EVENT SHALL CVS BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER.** Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein. This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer. This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

Distributed by:  
Beurer North America LP  
1 Oakwood Boulevard, Suite 255  
Hollywood, FL 33020, USA  
[www.shop-beurer.com](http://www.shop-beurer.com)

Assembled in China  
German Design and Engineering

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Sujeto a errores y cambios

---

The *Bluetooth*<sup>®</sup> word mark and logos are registered trademarks owned by *Bluetooth*<sup>®</sup> SIG, Inc. and any use of such marks by Beurer North America LP is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC.

Android is a trademark of Google LLC.