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Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

7. Troubleshooting Guide

Problem	Possible cause	Solution
The device does not switch on when the ON/OFF button is pressed.	The battery may be depleted.	Charge the battery. If the device still does not work, contact Customer Service.
Gel pads do not adhere to the skin.	The gel pads may be dirty or worn. Skin in the treatment area may be rough or not clean enough.	Clean the surface of the gel pads with a damp, lint-free cloth. If adhesiveness does not improve, replace gel pads. Clean the skin before treatment. Do not use lotions or oils. Shave area if necessary.
Cannot feel stimulation in treatment area.	The gel pads may be dirty or worn. The battery may be depleted.	Clean pads as described above. Charge battery.
Unpleasant sensation felt underneath gel pads.	Pads are worn and cannot evenly distribute current.	Replace the gel pads.
Skin in the treatment area turns red.	Redness can be caused by increased blood flow in the treatment area caused by the device; this is normal.	Redness should go away a few minutes after treatment. If redness persists, or if you experience any itching or inflammation, contact your doctor.
The Menstrual Relief becomes too hot.	Treatment duration too long.	Press the ON/OFF button to shut off the heat function and continue with TENS treatment only.

8. Technical Specifications

Model	EM 50 Menstrual Relief
Type	FT610-B
Output waveform	Biphasic rectangular pulse
Pulse length	100 µs
Pulse frequency	100 Hz
Output voltage	Max. 110 V _{pp} +/- 10 % (at 1000 Ω)
Heat output	109 °F (43 °C)
Output current	Max. 110 mA _{pp} +/- 10 % (at 1000 Ω)
Voltage supply	320 mAh lithium polymer battery
Treatment time	20 minutes to auto shutoff
Intensity Levels	15
Operating conditions	41° - 104° F (5° - 40° C) at 15 - 90% relative humidity
Storage conditions	32° - 104 °F (0° - 40° C) at 0 - 90 % relative humidity
Dimensions	3.7 x 7.6 x 0.6 inches (95 x 193 x 15 mm)
Weight	1.6 oz (47 g)

9. FCC Compliance Information

Menstrual Relief EM 50

Responsible Party – U.S. Contact Information
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020
United States
1-800-536-0366
info@beurer.com

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: Reorient or relocate the receiving antenna. Increase the separation between the equipment and receiver. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected. Consult the dealer or an experienced radio/TV technician for help. Warranty Your Beurer Menstrual Relief, model EM 50, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Menstrual Relief, model EM 50, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and you only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your cost in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope

of the limited lifetime warranty, you will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; old worn batteries; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturer's recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY, ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLICIT OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you. Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or ressealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state. For more information regarding our product line in the USA, please visit: www.beurer.com

Distributed by:
Beurer North America LP
1 Oakwood Boulevard, Suite 255
Hollywood, FL 33020, USA
www.beurer.com

Questions or comments? Call our US-based customer service toll free at 1-800-536-0366.

Assembled in China

Annex 1 – Electromagnetic Compatibility

A1.1 Important information regarding Electro Magnetic Compatibility (EMC)

With the increased number of electronic devices such as computers and mobile (cellular) telephones, this device in use may be susceptible to electromagnetic interference from other devices, even if it complies with CISPR emission requirements. Electromagnetic interference may result in incorrect operation of this device and create a potentially unsafe situation. Besides, this device should also not interfere with other devices. In order to regulate the requirements for EMC (Electro Magnetic Compatibility) with the aim to prevent unsafe product situations, the IEC 60601-1-2 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices. This device conforms to this IEC 60601-1-2 standard for both immunity and emissions. Nevertheless, special precautions need to be observed: The use of the device adjacent to or stacked with other device should be avoided because dot could result in improper operation. In case such use is necessary, the device and other devices should be observed to verify that they dare operating normally?
• The use of any parts other than those specified or provided could result inincreased electromagnetic emissions or decreased electromagnetic immunity of this deviceand result in improper operation.

• Portable RF communications device (including peripherals such as antenna cables and external antenna) should be used no closer than 12 inches (30 cm) to any part of this device. Otherwise, degradation of the performance of the device could result.

• Refer to further guidance below regarding the EMC environment in which the device shouldbe used.

A1.2 Guidance and manufacturer's declaration – Electromagnetic emission

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Test Standard	Test Item	Test Level	Electromagnetic environment – Guidance
EM 55011 / CISPR 11	Conducted Emissions of Powerline	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
EM 55011 / CISPR 11	Radiated Emissions	Class B	This device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
EN/IEC 61000-3-2	Harmonic Current Emission	Not applicable (Note 1)	–
EN/IEC 61000-3-3	Voltage Fluctuations and Flicker	Not applicable (Note 1)	–
Note 1:	The power supply of this device is from battery (DC voltage), so Harmonic current emission, Voltage fluctuations & flicker, EFT, Surge, CS and DIP tests are not applicable for this device.		
Note 2:	Standards applicable to other modes or EM ENVIRONMENTS of transportation for which use is intended shall apply.		

A1.3 Guidance and manufacturer's declaration – Electromagnetic immunity

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Test Standard	Test Item	Test Level	Electromagnetic environment – Guidance
IEC 61000-4-2	Electrostatic discharge	Contact: ± 8 kV, Air-±2 kV, ±4 kV, and ±5 kV	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%. Portable and mobile RF communications equipment should be used no closer than 30 cm (12 inch) to any part of this device.
IEC 61000-4-3	Radiated RF EM fields	10 V/m; 80 MHz to 2.7 GHz; 80% AM at 1 kHz	Portable and mobile RF communications equipment should be used no closer than 30 cm (12 inch) to any part of this device.
IEC 61000-4-3	Proximity fields from RF wireless communications equipment	Not applicable (Note 1)	–