ENGLISH	Digital EMS/TENS Unit	
	Instructions for use	2
ESPAÑOL	Dispositivo digital EMS/TENS	
	Instrucciones de uso.....	60
FRANÇAIS	Appareil EMS/TENS numérique	
	Mode d'emploi	60

**READ THIS MANUAL COMPLETELY AND CAREFULLY
BEFORE USING THIS PRODUCT**
Keep this manual in a safe location for future reference

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SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE

If the instruction manual is damaged or if you no longer have the instruction manual in your possession, please contact Beurer customer service. Please see warranty for service contact.

Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for application in the areas of heating, weight management, blood pressure, body temperature, pulse, gentle therapy, massage, and beauty.

Register your product today to receive further benefits:

- added protection for your purchase
- health and wellness tips
- new product information
- continual outstanding customer service and support.

Visit www.registerbeurer.com or contact customer service at 1-800-536-0366.

Please read these instructions for use carefully and keep them for later use. Make them accessible to other users and observe the information they contain.

With kind regards,
Your Beurer Team

1. IMPORTANT SAFETY INSTRUCTIONS

Signs and symbols

Whenever used, the following signs identify safety and property damage messages and designate a level of hazard or seriousness.

READ AND UNDERSTAND THIS ENTIRE MANUAL INCLUDING THE SAFETY SECTION AND ALL INSTRUCTIONS AND WARNINGS COMPLETELY AND CAREFULLY BEFORE USING THIS PRODUCT. FOLLOW ALL SAFETY INSTRUCTIONS AND WARNINGS TO AVOID HAZARDOUS SITUATIONS AND TO MAKE CORRECT USE OF THIS PRODUCT.

	This is the safety alert symbol. It alerts you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury or death.
 WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury.
 CAUTION	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury.
NOTICE	NOTICE addresses practices not related to personal injury, such as product and/or property damage.

WARNING

The device is not a substitute for medical consultation and treatment. Consult your doctor first if you are experiencing any pain or are suffering from an illness.

Only use digital EMS/TENS units:

- On people.
- For external use.
- For the purpose for which it was developed and in the manner specified. Any improper use can be hazardous.
- With the original accessories supplied, which can be re-ordered. Failure to do so invalidates the warranty.

Contraindications

Do not use this device if you have any of the following:

- Implanted electrical devices (e.g. pacemakers).
- Metal implants.
- A known or acute cardiac arrhythmia, or disorders of the heart's impulse and conduction system.
- Acute illness.
- If you suffer from a seizure disorder (e.g. epilepsy).
- Malignant diseases.
- Skin injuries or cuts.
- Pregnancy.
- An increased tendency to bleed, e.g. after an acute injury or a fracture. The stimulation current can trigger or exacerbate bleeding.
- Recent operation when muscle contractions could impair the healing process.
- A high frequency surgical device connected.
- Low or high blood pressure.
- High fever.
- Psychosis.
- Swollen or inflamed regions.
- An insulin pump.

- Cancer (the effects of electrical stimulation on cancerous tissue are unknown).
- Acute or chronic diseases of the gastrointestinal tract.
- A tendency to hemorrhage following acute trauma or fracture.

To avoid personal injury, do not use this device:

- On the neck/carotid artery, as it could cause cardiac arrest.
- Across or through the head, directly on the eyes, covering the mouth, on the front of the neck, (especially the carotid sinus), or with electrodes placed on the chest and the upper back or crossing over the heart.
- Near the upper chest, since it could increase the risk of ventricular fibrillation leading to cardiac arrest.
- In the genital area.
- On acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin – whether painful or not, reddened skin, rashes, e.g. allergies, burns, bruises, swellings, both open and healing wounds, and post-operative scars where the healing process could be affected) or over swollen, red or inflamed areas or skin eruptions (e.g. phlebitis, thrombophlebitis, varicose veins).
- On areas of injury or restricted movement (e.g., fractures or sprains).
- If you are under medical supervision for cognitive dysfunction as you may not be able to comply with safety instructions.
- When asleep, when driving a vehicle or when operating machinery.
- If connected to a high-frequency surgical device.
- After consuming alcohol.
- Whilst undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous.
- At the same time as other devices that emit electrical pulses.
- The device must never be used near to the heart. The stimulation electrodes must not be placed on any part of the front ribcage (where the ribs and breastbone are located), especially not on the two large pectorals. This can increase the risk of ventricular fibrillation and induce cardiac arrest.
- While you bathe or shower. Do not store or keep the device where it can fall into a bathtub or sink. Do not use the device in humid environments.
- While using any life keeping electronic medical devices, TENS devices, electrocardiograph and any other medical screening and monitoring devices.
- On the skeletal skull structure, or around the mouth, throat or larynx.
- If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.

- Ensure that no metallic objects (e.g. belt buckles or necklaces) come into contact with the electrodes during stimulation. If you are wearing jewellery or have piercings in the area to be treated (e.g. a navel piercing), these must be removed before using the device. Failure to do so could result in spot burns.

Before using the device, consult your doctor if any of the following applies to you:

- Serious illnesses, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or recurrent malignant growths.
- Any skin conditions.
- Undiagnosed chronic pain.
- Diabetes.
- Any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- If you are receiving medical treatment.
- In the event of complaints linked to stimulation treatment.
- If you suffer from persistently irritated skin due to long-term stimulation at the same electrode site
- If in doubt.
- After a recent surgical procedure; stimulation may disrupt the healing process.
- After delivering a baby.
- After an IUD contraceptive device (e.g., coil) has been fitted.
- After the heavy days of your period have finished, as vigorous abdominal exercise is not recommended at this time.

 **WARNING**

- The device is only for private use and is not intended for medical or commercial purposes.
- The device is only designed for use by healthy adults. Keep the device away from children.
- Keep the device away from heat sources.
- Never use the device near microwave ovens or microwave devices (e.g. cell phones) as they can affect operation.
- Never immerse the device in water or other liquids.
- Check the device regularly for wear and damage. If you find such signs, discontinue use and contact customer service.
- Switch off the device immediately if it malfunctions.
- Do not attempt to repair the device yourself. Contact customer service for all repairs.
- The long-term effects of chronic electrical stimulation are unknown.

- Carry out the first few minutes of treatment while sitting or lying down, so that if you experience a rare vasovagal reaction (i.e., you feel faint) then you are not at unnecessary risk of injury. If you start to feel faint, shut off the unit immediately and elevate your legs for approximately 5 – 10 minutes.

CAUTION

- An effective session should not cause discomfort. Do not overexert yourself while using the device. Any session should be at a comfortable level for you.
- For first time users, muscle stimulation can be an unusual sensation. It is recommended to begin in a seated position with low stimulation intensity settings to familiarize yourself with the sensation before progressing to higher intensity settings.
- Always pull firmly on electrodes to remove them from the skin to prevent injuries in the unusual case of highly sensitive skin.
- The device is suitable for self-treatment.
- If you feel unwell or experience pain, stop using it immediately.
- Switch off the unit or the respective channel first before removing or relocating electrodes to prevent unintentional stimulation.
- Do not modify electrodes (e.g. by cutting them). This increases the current intensity, which is potentially hazardous (max. recommended output value for the electrodes is 9 mA/cm², an effective current intensity beyond 2 mA/cm² requires increased awareness).
- Make sure not to confuse the electrode cables including contacts with your headphones or other devices and do not connect the electrodes to other devices.
- Do not use in the vicinity of highly flammable substances, gases or explosives.
- Treatment of the skin with moisturizing lotions or ointments beforehand is not recommended as this considerably increases the electrode wear and may cause unpleasant current peaks.
- This device is not intended for use by children or people with restricted physical, sensory (e.g. reduced sensitivity to pain) or mental skills or a lack of experience and/or lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device.
- If the adhesive capability of the adhesive electrodes decreases, please replace them immediately. Only use the device again with new adhesive electrodes. Otherwise the unequal adhesion of the adhesive electrodes may lead to skin injuries. Replace the electrodes with new ones after having used them 20 times at the latest.
- In order to perform the effectiveness of function by the device, do not drop the device or disassemble the device.
- In rare instances, skin irritation in the region of the electrodes may occur.
- For hygienic reasons, use the device only on one person.
- Do not use the product in a bathroom or a moist environment.

- Do not store or keep the device where it can fall into a bathtub or sink.
- Do not expose the device to direct sunlight or high temperatures.
- Protect the device from dust, dirt and humidity.

NOTICE:

- Do not insert the electrode plug into any place other than the jack on the main unit.
- Do not pull on the electrode cord during treatment.
- Do not use the device while wearing electronic devices such as watches as this may damage the device.
- When pulling out the cord from the device, hold the plug and pull.

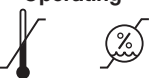
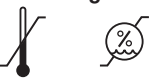
Battery Handling Safety Precautions

- Use only the size and type of batteries specified.
- Be sure to follow the correct polarity when installing the batteries. Reversed batteries may cause damage to the device.
- Do not mix different types of batteries together (e.g. Alkaline and Carbon-zinc or rechargeable batteries) or old batteries with fresh ones. Always replace batteries as a simultaneous set.
- If the batteries in the device are depleted or the device will not be used for a long period of time, remove the batteries to prevent damage or injury from possible battery leakage.
- Do not try to recharge batteries not intended to be recharged; they can overheat and rupture (follow battery manufacturer's directions.)
- Do not dispose of batteries in fire, batteries may explode or leak.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove discharged batteries from the product and dispose/recycle in compliance with all applicable laws.
- Keep batteries away from children and pets. Batteries may be harmful if swallowed. Should a child or pet swallow a battery, seek medical assistance immediately.
- Do not use rechargeable batteries.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- Do not disassemble, open or crush the batteries.
- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.

2. Important information

Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Follow instructions for use
	Do not use this device with a pacemaker
	Application part, type BF
	Serial Number
IP22	Device protected against foreign objects ≥ 12.5 mm and against water dripping at an angle up to 15° .
	The device can emit effective output values above 10 mA, averaged over every five-second interval.
Operating 	Permissible operating temperature and humidity.
Storage 	Permissible transport and storage temperature. Permissible transport and storage humidity.

3. Getting to know your device

What is a digital EMS/TENS unit and what can it do?

Digital EMS/TENS units are electrostimulation devices. This unit provides two basic functions suitable for combined operation:

1. Electrical stimulation of nerve tracts (TENS)
2. Electrical stimulation of muscle tissue (EMS)

The unit also features two independent stimulation channels and four self-adhesive electrodes. It offers a wide range of functions for increasing general well-being, pain relief, maintaining physical fitness, muscle revitalization, and combating tiredness. For these purposes, you can either choose from preset programs or specify your own to suit your individual needs. The principle of electrostimulation is based on the imitation of impulses in our bodies that are transferred to nerve and muscle fibers using electrodes. The electrodes can be applied to many parts of the body and are completely harmless and virtually painless. In certain applications, you may notice a slight tingling or vibrating sensation. The electrical impulses that are sent into the tissue influence the transmission of stimulation into nerves, nerve centers, and muscle groups in the application area. Electrostimulation usually only has an effect after regular application and does not replace regular training. However, it is a sensible, supplementary training element.

TENS, or transcutaneous electrical nerve stimulation, relates to the electrical stimulation of the nerves via the skin. TENS is an effective non-pharmacological method of treating different types of pain from a variety of causes and has no side effects if administered correctly. This method has been clinically tested and approved and can be used for simple self-treatment. The pain-relieving or pain-suppressing effect is achieved by inhibiting the transference of pain to nerve fibers (caused mainly by high-frequency impulses) and by increasing secretion of endorphins in the body, which reduces the sensation of pain. This method is scientifically substantiated and approved as a form of medical treatment.

Any symptoms that can be relieved using TENS must be checked by your doctor, who will also instruct you on how to perform TENS self-treatment. TENS is clinically tested and approved to treat the following issues:

- Back pain, particularly in the lumbar/cervical spine area.
- Sore joints (e.g. knee, hip and shoulder joints).
- Temporary relief of pain associated with sore and aching muscles in the shoulder, waist, back, neck, upper extremities (arm), lower extremities (leg), abdomen and bottom due to strain from exercise or normal household work activities.


Electrical muscle stimulation (EMS) is a widespread and generally recognized method that has been used in sports medicine and rehabilitation for years. In sports and fitness, EMS is used to complement conventional muscle training, increase the performance of muscle groups, and adjust physical proportions to achieve desired aesthetic results.

EMS activating application involves:

- Muscle training to increase endurance.
- Muscle training to support the strengthening of specific muscles or muscle groups, and to achieve the desired changes to physical proportions.

4. Package Contents

- 1 x digital EMS/TENS unit
- 1 belt clip (A)
- 2 x connection cables (B)
- 4 x electrodes (C); size: 1.77 x 1.77 in (45 x 45 mm)
- 3 x 1.5 V AAA batteries (D)
- Instructions for use

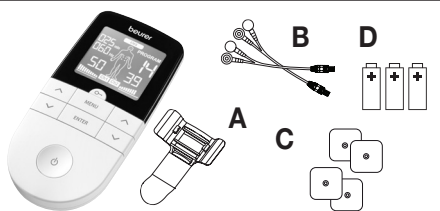
5. Parts and Controls

- 1 ON/OFF button
- 2 **ENTER** button
- 3 Setting buttons (**Ch1** left, **Ch2** right)
- 4 **MENU** button
- 5 Keylock

Display description

- 1 Menu **TENS** / **EMS**
- 2 Program number
- 3 Impulse intensity channel 2 (**Ch2**)
- 4 Electrode positioning indicator
- 5 Impulse intensity channel 1 (**Ch1**)
- 6 Low battery
- 7 Keylock
- 8 Display for frequency (Hz) and pulse width (μ s)
- 9 Timer function (remaining time display) or working time display)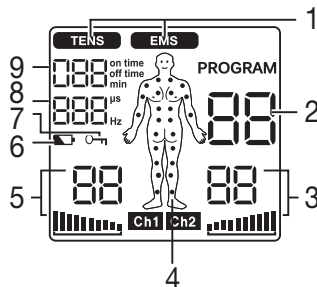

6. Initial Use

1. Remove the belt clip from the unit (if attached).
2. Press the battery compartment cover on the rear of the device and slide it downwards to remove.
3. Insert three 'AAA' size alkaline batteries (included), following the polarity markings inside the battery compartment.
4. Reattach the battery compartment cover (Fig. 1).
5. Reattach the belt clip, if desired.
6. Connect the connection cables with the electrodes (Fig. 2). The electrodes feature clip closures for easier connection.
7. Guide the connection cable plugs into the socket on the top of the device (Fig. 3).
8. Do not pull, twist, or make sharp bends in the cables (Fig. 4).

i When the batteries are removed, all settings are restored to the factory default settings.

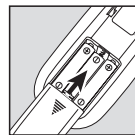
Fig. 1

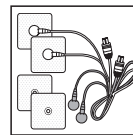
Fig. 2

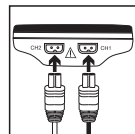
Fig. 3

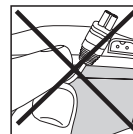
Fig. 4

7. To Use

NOTES:

- The device shuts itself off if not used for one minute (automatic shutoff). When the unit is switched on again, the LCD screen displays the menu selection and the most recently used menu flashes.
- The device will beep once when a valid button is pressed or twice when an invalid button is pressed.
- To pause treatment at any time, press the ON/OFF button . To continue, press the ON/OFF Button again and reset the desired impulse intensity.
- Both pads of the channel used needs to be properly attached to your body, in order for the device to work.

Starting Application

Step 1: Find a suitable program in the program tables (see the 'Program Overview' section).

Step 2: Place electrodes on the desired treatment area (for positioning suggestions see 'Information regarding the positioning of electrodes') and connect them to the device.

Step 3: Press the ON/OFF button  to switch the device on.

Step 4: Press the **MENU** button to navigate through the **TENS** / **EMS** menus and press the **ENTER** button to confirm.

Step 5: Use the **Λ/V** setting buttons to select a program number and press the **ENTER** button to confirm. At the start of the stimulation treatment, the impulse intensity of **Ch1** and **Ch2** is set to 00 by default. No impulses are sent to the electrodes yet.

Step 6: Use the left and right **Λ/V** buttons for **Ch1** and **Ch2** to select an impulse intensity; the indicator for impulse intensity in the display changes accordingly. Keep increasing the intensity until you start feeling it. You can always increase the intensity during treatment. If a program is paused, the intensity cannot be increased.

General information

To return to the previous selection menu, press the **MENU** button. Press and hold the **ENTER** button to skip individual setting steps and start directly with the stimulation treatment.

Keylock

Locks the buttons to avoid them being pressed unintentionally.

1. To activate the keylock, press and hold the Keylock button  for about three seconds until the key icon displays.
2. To deactivate, press and hold the Keylock button  again for about three seconds until the key icon disappears.

Pausing Application

To pause the stimulation at any time, press the ON/OFF button . To continue, press the ON/OFF Button again and reset the desired impulse intensity.

8. Program Overview

What is a digital EMS/TENS unit and what can it do?

- 15 TENS programs
- 35 EMS programs

In all programs, you can set the impulse intensity of both channels individually. You can also set various parameters in TENS programs 13-15 and EMS programs 33-35 to adjust the stimulating effect to the application area.

TENS Program Table

Prog. No.	Area of application, indications	Running time (min)	Possible electrode positions
1	Pain in upper limbs 1	30	12-17
2	Pain in upper limbs 2	30	12-17
3	Pain in lower limbs	30	23-27
4	Ankle pain	30	28
5	Shoulder pain	30	1-4
6	Pain in the back	30	4-11
7	Pain in bottom and back of thighs	30	22, 23
8	Pain relief 1	30	1-28
9	Pain relief 2	30	1-28
10	Endorphin effect (burst)	30	1-28
11	Pain relief 3	30	1-28
12	Pain relief – chronic pain	30	1-28

NOTE:

- TENS programs 13-15 can be set individually (see the 'Customizable Programs' section).
- See the 'Electrode Positioning' section for correct electrode placement.

EMS Program Table

Prog. No.	Area of application, indications	Running time (min)	Possible electrode positions
1	Warmup	30	1-27
2	Capillarization	30	1-27
3	Strengthening the upper arm muscles	30	12-15
4	Maximizing the strength of the upper arm muscles	30	12-15
5	Explosive force of the upper arm muscles	30	12-15
6	Tightening the upper arm muscles	30	12-15
7	Shaping the upper arm muscles	30	12-15
8	Tightening the lower arm muscles	30	16-17
9	Maximizing the strength of the lower arm muscles	30	16-17
10	Shaping the lower arm muscles	30	16-17
11	Tightening the abdominal muscles	30	18-20
12	Maximizing the strength of the abdominal muscles	30	18-20
13	Shaping the abdominal muscles	30	18-20
14	Toning the abdominal muscles	30	18-20
15	Strengthening the thigh muscles	30	23, 24
16	Maximizing the strength of the thigh muscles	30	23, 24
17	Explosive force of the thigh muscles	30	23, 24
18	Shaping the thigh muscles	30	23, 24
19	Toning the thigh muscles	30	23, 24
20	Strengthening the lower leg muscles	30	26, 27
21	Maximizing the strength of the lower leg muscles	30	26, 27
22	Explosive force of the lower leg muscles	30	26, 27
23	Shaping the lower leg muscles	30	26, 27
24	Toning the lower leg muscles	30	26, 27
25	Strengthening the shoulder muscles	30	1-4
26	Maximizing the strength of the shoulder muscles	30	1-4
27	Tightening the shoulder muscles	30	1-4
28	Strengthening the lower back muscles	30	4-11

Prog. No.	Area of application, indications	Running time (min)	Possible electrode positions
29	Maximizing the strength of the lower back muscles	30	4-11
30	Tightening the gluteal muscles	30	22
31	Strengthening the gluteal muscles	30	22
32	Maximizing the strength of the gluteal muscles	30	22

NOTE:


- EMS programs 33-35 can be set individually (see the 'Customizable Programs' section).
- See the 'Electrode Positioning' section for correct electrode placement.

NOTE: See the 'Electrode Positioning' section for correct electrode placement

WARNING: Do not apply the electrodes to the pectoral muscles (chest).

9. Electrode Positioning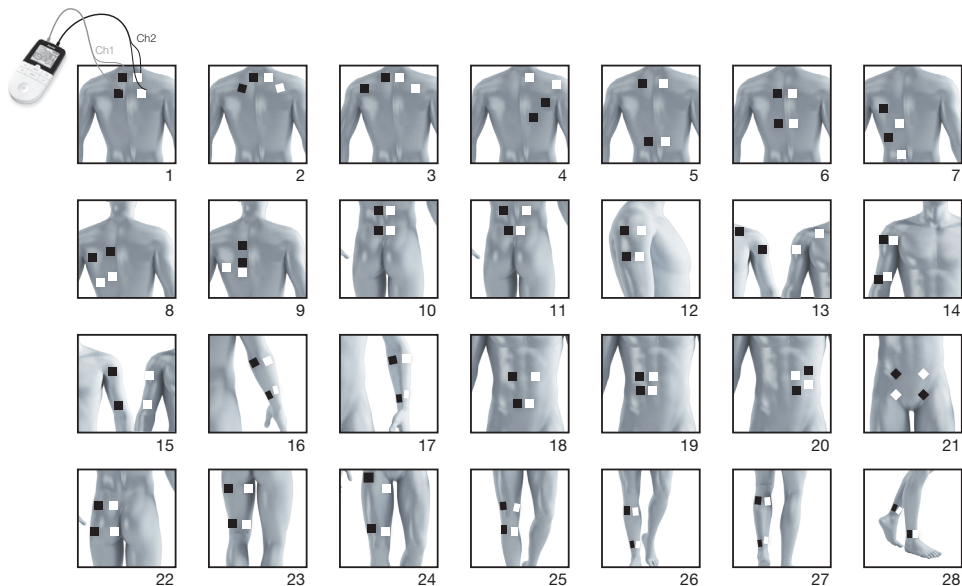

It is crucial to position the electrodes properly. We recommend that you consult your doctor to establish the ideal electrode positions for your intended application area.

The figure on the display is intended as an initial aid to help you position the electrodes.


Electrode Distance

The larger the distance between electrodes, the larger the stimulated tissue volume. This applies to the area and depth of the tissue volume. At the same time, however, the stimulation intensity of the tissue falls the further the electrodes are apart. As a result, greater distances between electrodes mean a larger tissue volume is stimulated, but less intensively. Consequently, you must increase the impulse intensity to boost stimulation.

- Proper distance between electrodes: 5-15 cm (2-6 in)
- At distances less than 5 cm (2 in), the unit primarily stimulates surface structures intensively.
- At distances more than 15 cm (6 in), large areas and deep structures are stimulated very weakly.

Relationship between electrodes and muscle fiber structures

Adapt the current flow direction to the fiber structure of the muscle according to the muscle layer you would like to treat. If you are targeting surface muscles, position the electrodes in parallel to the fiber structure (A – B / C – D) and if you are targeting deeper layers of tissue, position the electrodes across the fiber structure. You can do this by positioning electrodes as crosses (i.e. diagonally), such as A-D/B-C.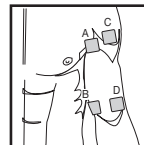

NOTE: As part of pain relief treatment (TENS) using separately adjustable channels and two adhesive electrodes each, either position both electrodes of a channel so the area affected by pain is between the electrodes or position one electrode directly on the area affected by pain and the other electrode at least 2-3 cm (1-1½ in) away.

You may use the electrodes of the second channel to simultaneously treat additional areas affected by pain or use them in conjunction with the electrodes of the first channel to restrict the area affected by pain (position electrodes opposite). In this case, position electrodes as crosses.

NOTE:

- Use the electrodes on skin that is clean and preferably free from hair and grease in order to prolong the life of the electrodes. For best results, clean the skin with water and remove hair prior to treatment.
- If an electrode should come loose during use, the impulse intensity of both channels drops to the lowest level. Apply the electrode again and reset the desired impulse intensity.

10. Customizable Programs

(TENS programs 13-15, EMS programs 33-35)

TENS programs 13-15 and EMS programs 33-35 can be customized according to your needs.

TENS 13

In this program, you can set the impulse frequency between 1 and 150 Hz and the impulse width between 80 and 250 μ s.

1. Place the electrodes on the desired area for treatment (for placement suggestions see the 'Electrode Positioning' section) and connect them to the device.
2. Select the TENS 13 program as described in the 'Starting Application' section, steps 3-5.
3. Use the Δ/V setting buttons to select an impulse frequency and use the **ENTER** button to confirm.
4. Use the Δ/V setting buttons to select an impulse width and use the **ENTER** button to confirm.
5. Use the Δ/V setting buttons to select a treatment time and use the **ENTER** button to confirm.
6. Use the left and right Δ/V setting buttons for **Ch1** and **Ch2** to select an impulse intensity.

TENS 14

Various impulse sequences run in this **burst** program. Burst programs are suitable for all areas of application to be treated with changing signal patterns (to minimize the level of accustoming to the treatment). In this program, you can set an impulse width between 80 and 250 μ s.

1. Place the electrodes on the desired area for treatment (for placement suggestions see the 'Electrode Positioning' section) and connect them to the device.
2. Select the TENS 14 program as described in the 'Starting Application' section, steps 3-5.
3. Use the Δ/V setting buttons to select an impulse width and use the **ENTER** button to confirm.
4. Use the Δ/V setting buttons to select a treatment time and use the **ENTER** button to confirm.
5. Use the left and right Δ/V setting buttons for **Ch1** and **Ch2** to select an impulse intensity.

TENS 15

In this program, you can set the impulse frequency between 1 and 150 Hz. The impulse width changes during the stimulation treatment.

1. Place the electrodes on the desired area for treatment (for placement suggestions see the 'Electrode Positioning' section) and connect them to the device.
2. Select the TENS 15 program as described in section 'Starting Application' section, steps 3-5.
3. Use the Δ/V setting buttons to select an impulse frequency and use the **ENTER** button to confirm.
4. Use the Δ/V setting buttons to select a treatment time and use the **ENTER** button to confirm.
5. Use the left and right Δ/V setting buttons for **Ch1** and **Ch2** to select an impulse intensity.

EMS 33

In this program, you can set the impulse frequency between 1 and 150 Hz and the impulse width between 80 and 320 μ s.

1. Place the electrodes on the desired area for treatment (for placement suggestions see the 'Electrode Positioning' section) and connect them to the device.
2. Select the EMS 33 program as described in the 'Starting Application' section, steps 3-5.
3. Use the Δ/V setting buttons to select an impulse frequency and use the **ENTER** button to confirm.
4. Use the Δ/V setting buttons to select an impulse width and use the **ENTER** button to confirm.
5. Use the Δ/V setting buttons to select a treatment time and use the **ENTER** button to confirm.
6. Use the left and right Δ/V setting buttons for **Ch1** and **Ch2** to select an impulse intensity.

EMS 34

In this program, you can set the impulse frequency between 1 and 150 Hz and the impulse width between 80 and 450 μ s. You can also set the working time and pause time for this program between 1 and 30 seconds each.

1. Place the electrodes on the desired area for treatment (for placement suggestions see the 'Electrode Positioning' section) and connect them to the device.
2. Select the EMS 34 program as described in the 'Starting Application' section, steps 3-5.
3. Use the Δ/V setting buttons to select the working time ('on time') and use the **ENTER** button to confirm.
4. Use the Δ/V setting buttons to select the pause time ('off time') and use the **ENTER** button to confirm.
5. Use the Δ/V setting buttons to select an impulse frequency and use the **ENTER** button to confirm.
6. Use the Δ/V setting buttons to select an impulse width and use the **ENTER** button to confirm.
7. Use the Δ/V setting buttons to select the treatment time and use the **ENTER** button to confirm.

8. Use the left and right \wedge/\vee setting buttons for **Ch1** and **Ch2** to select an impulse intensity.

EMS 35

Various impulse sequences run in this burst program. Burst programs are suitable for all areas of application to be treated with changing signal patterns (to minimize the level of accustoming to the treatment). In this program, you can set the impulse frequency between 1 and 150 Hz and the impulse width between 80 and 450 μs . You can also set the working time and pause time for this program between 1 and 30 seconds each.

1. Place the electrodes on the desired area for treatment (for placement suggestions see the 'Electrode Positioning' section) and connect them to the device.
2. Select the EMS 35 program as described in the 'Starting Application' section, steps 3-5.
3. Use the \wedge/\vee setting buttons to select the working time ('on time') and use the **ENTER** button to confirm.
4. Use the \wedge/\vee setting buttons to select the pause time ('off time') and use the **ENTER** button to confirm.
5. Use the \wedge/\vee setting buttons to select an impulse frequency and use the **ENTER** button to confirm.
6. Use the \wedge/\vee setting buttons to select an impulse width and use the **ENTER** button to confirm.
7. Use the \wedge/\vee setting buttons to select the treatment time and use the **ENTER** button to confirm.
8. Use the left and right \wedge/\vee setting buttons for **Ch1** and **Ch2** to select an impulse intensity.

11. Doctor's Function

The Doctor's Function is a special setting to allow you to access your personal program even more easily and directly. Your individual program settings are instantly recalled and activated when the device is switched on. You may wish to adjust this individual program following advice from your doctor.

Setting the Doctor's Function

1. Select your program and the corresponding settings as described in the 'Starting Application' section.
2. At the start of the stimulation treatment, the impulse intensity of **Ch1** and **Ch2** is set to 00 by default. No impulses are sent to the electrodes yet. Before setting the desired impulse intensity using the intensity setting buttons, press and hold the **Ch2** \vee button for five seconds. Storage in the Doctor's function is confirmed with a long beep.
3. If you switch on the device again, the program you saved using the Doctor's function opens directly.

Deleting the Doctor's Function

To clear the device and allow access to other programs again, press and hold the **Ch2** ∇ button again for about five seconds. To do this, the impulse intensity of **Ch1** and **Ch2** must be set to 00.

Deletion of the Doctor's function is confirmed with a long beep.

Therapy Memory

This device will record your treatment time. To access the therapy memory, power the device on and then press and hold the **Ch2** \blacktriangle button for five seconds. The elapsed treatment time appears in the display. The top two numbers are the minutes, with the hours shown underneath. To reset the treatment time, press and hold the **Ch2** ∇ button for five seconds. When replacing the battery, the therapy memory is reset. Press the Menu button to return to selecting a program, or shut the device off.



NOTE: Therapy memory cannot be accessed if the Doctor's Function is activated.

12. Current Parameters

Electrostimulation units operate with the following current settings that may affect stimulation effects differently:


Impulse shape

Describes the time function of the electrical impulse. It distinguishes between monophasic and biphasic pulse currents. In monophasic pulse currents, the current flows in one direction and in biphasic pulse currents, the electrical impulse alternates direction. The digital EMS/TENS unit only provides biphasic pulse currents as these relieve muscles, cause little muscle fatigue, and provide safer application.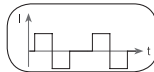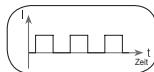


Impulse frequency

Indicates the number of impulses each second and is shown in Hz (Hertz). It can be calculated by determining the cyclic value for the time period. The relevant frequency determines which types of muscle fibers react favorably. Slow-reacting fibers react more easily to lower impulse frequencies up to 15 Hz, whereas fast-reacting fibers only respond from approximately 35 Hz and up. Impulses around the 45-70 Hz range are linked with constant muscle tension and quicker fatigue. Higher impulse frequencies are therefore favorable in strength and maximum power training.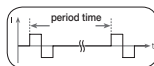


Impulse width

Indicates the duration of an individual impulse in microseconds. The impulse width therefore determines, among other things, the penetration of the electricity; larger muscle masses usually require larger impulse widths.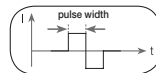

Impulse intensity

Setting the intensity levels depends on the individual sensitivity of each user and is determined by a variety of variables, such as site of application, blood circulation to the skin, skin thickness, and the quality of the electrode contact. The setting used should be effective but should never cause pain at the site of application. While a gentle tingling indicates sufficient stimulation energy levels, any setting that causes pain should be avoided. Longer applications may require readjustments due to the adjustment processes over time at the application site.

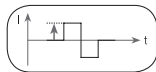Cycled impulse parameter variation

In many cases, it is necessary to cover the overall tissue structure at the site of application by applying several impulse parameters. In the digital EMS/TENS unit, this is achieved by the provided programs that automatically make a cyclical impulse parameter change. This also prevents individual muscle groups at the site of application being affected by fatigue. This digital EMS/TENS unit provides sensible default current parameter settings. With this, you can change the impulse intensity at any time during use. For six programs, you can also set various parameters for stimulation yourself.

13. Care and Maintenance

Adhesive electrodes

- To maximize the life of the electrode adhesive, clean them carefully with a damp, lint-free cloth or clean the underside of the electrodes under warm running water and pat dry with a lint-free cloth.
- Before cleaning with water, remove the connection cables from the electrodes.
- Reapply the electrodes to the carrier foil following treatment.

Cleaning the device

- Shut the device off before cleaning.
- Remove the batteries from the device before cleaning.
- Clean the device after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution.
- Do not use any chemical or abrasive cleaning agents.

 Ensure that no water enters the device.

Storage

- Remove the batteries from the device if you will not be using it for a prolonged period. Leaking batteries may damage the device.
- Do not make sharp bends in the connection cables and electrodes.
- Disconnect the connection cables from the electrodes.
- Reapply the electrodes to the carrier foil after use.
- Store the device and accessories in a cool, well-ventilated space.
- Never place any heavy objects on the device.

Disposal

Observe the local regulations for material disposal. Dispose of the device in accordance with local regulations. If you have any questions, please contact the local authorities responsible for waste disposal.

14. Troubleshooting

The device does not switch when the ON/OFF button is pressed.

- (1) Ensure batteries have been inserted correctly and are in contact with the terminals.
- (2) Replace batteries, if applicable.
- (3) Contact Customer Services.

Electrodes do not adhere to the body.

- (1) Clean the adhesive surface of the electrodes using a damp, lint-free cloth. Replace the electrodes if they still do not adhere securely.
- (2) Clean the skin prior to any application; do not use skincare lotions or oils prior to treatment. Shaving may increase the life of electrodes.

There is no noticeable stimulation.

- (1) Press the ON/OFF button to interrupt the program. Check the connection cables are correctly connected to the electrodes. Ensure the electrodes are in firm contact with the treatment area.
- (2) Ensure the connection plug is firmly connected to the device.
- (3) Press the ON/OFF button to restart the program.

- (4) Check electrode positions and ensure that adhesive electrodes do not overlap.
- (5) Gradually increase the impulse intensity.
- (6) The batteries are almost empty. Replace the batteries.

The low battery symbol is shown.

Replace all batteries.

You have an unpleasant sensation at the electrodes.

- (1) The electrodes are not positioned correctly. Check their positions and re-position, if necessary.
- (2) The electrodes are worn. This may cause irritated skin as even distribution of the current across the entire area is no longer guaranteed. For this reason, replace the electrodes.

Skin in the treatment area turns red.

Immediately stop treatment and wait until your skin has returned to its normal condition. If the redness is under the electrode and disappears quickly, there is no risk, as this is caused by the locally stimulated, increased blood flow. Consult your doctor, however, before you continue treatment if the skin irritation persists and is accompanied by an itchy sensation or inflammation since this may be caused by an allergic reaction to the adhesive surface.

15. Specifications

Model:	EM 49
Type:	EM 49-2
Output waveform	Biphasic rectangular pulse
Pulse length	50 - 450 μ s
Pulse frequency	1 - 150 Hz
Output voltage max.	100 Vpp (500 ohm)
Output current max.	200 mApp (500 ohm)
Voltage supply	3 AAA batteries
Treatment time	Adjustable from 5 to 100 minutes
Intensity	Adjustable from 0 to 50
Operating conditions	41° - 104° F (5° - 40° C) at a relative humidity of 15 - 90%
Storage conditions	32° - 104°F (0°C - 40°C) at a relative humidity of 0 - 90%

Dimensions	5.2 x 2.5 x 1.1 in (132 x 63 x 29.5 mm) including belt clip
Weight	2.9 oz (83 g) without batteries, 4.1 oz (117 g) with batteries
Use altitude limit	9842 ft (3000 m)
Use range atmospheric pressure	700 – 1060 hPa

Subject to errors and changes. The serial number can be found on the unit itself or in the battery compartment.

This device complies with European standards IEC60601-1 and IEC60601-1-2 and is subject to special precautionary measures with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this unit. More details can be requested from the stated Customer Service address.

Notes on electromagnetic compatibility

WARNING!

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the devices electromagnetic immunity; this can result in faulty operation.
- Keep portable RF communication devices (including peripheral equipment, such as antenna cables or external antennas) at least 30 cm away from all device parts, including all cables included in delivery. Failure to comply with the above can impair the performance of the device.
- Failure to comply with the above can impair the performance of the device.

16. Replacement Parts

You can obtain the following replacement parts directly from customer service:

Description	Item number
-------------	-------------

4 adhesive electrodes 1.77 x 1.77 in (45 x 45 mm)

661.74

17. FCC Compliance Information

TENS/EMS device EM 49

Responsible Party U.S. Contact Information:

Beurer North America LP

1 Oakwood Boulevard, Suite 255,

Hollywood, FL 33020, United States

1-800-536-0366

info@beurer.com

FCC Compliance Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
 2. This device must accept any interference received, including interference that may cause undesired operation.
- Changes or modifications not expressly approved by the party responsible for compliance could void the users authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

18. Warranty

Limited Lifetime Warranty For Original Purchaser

Your Beurer Digital EMS/TENS device, Model EM 49, is warranted to be free from defects in materials and workmanship for the life of the product under normal conditions of intended use and service. This warranty extends only to the original retail purchaser and does not extend to retailers or subsequent owners.

We will, at our option, repair or replace the Beurer Digital EMS/TENS device, Model EM 49, without additional charge, for any part or parts covered by these written warranties. No refunds will be given. Repair or replacement is our only responsibility and your only remedy under this written warranty. If replacement parts for defective materials are not available, Beurer reserves the right to make product substitutions in lieu of repair or replacement.

For warranty service contact our customer service department at 1-800-536-0366 or at info@beurer.com to provide a description of the problem. If the problem is deemed to be within the scope of the limited lifetime warranty, you will be asked to mail the product at your costs in its original package with proof of purchase, your name, address and phone number. If the problem is not deemed to be within the scope of the limited lifetime warranty, we will provide a quotation for repair respectively replacement and return shipping fee.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of unauthorized accessory; alteration to the product; improper installation; misapplication; lack of reasonable care with respect to the product; unauthorized repairs or modifications; improper use of electrical/power supply; normal wear; loss of power; dropped product; malfunction or damage of an operating part as a result of failure to comply with instructions for use or to provide manufacturers recommended maintenance; transit damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Beurer. This warranty is void if the product is ever used in a commercial or business environment. The maximum liability of Beurer under this warranty is limited to the purchase price actually paid by the customer for the product covered by the warranty, as confirmed by proof of purchase, regardless of the amount of any other direct or indirect damage suffered by the customer.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaptation to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED IN DURATION TO THE DURATION OF THIS APPLICABLE WRITTEN WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL BEURER BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS, IMPLIED OR ANY OTHER THEORY OF LIABILITY, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

Beurer does not authorize anyone, including, but not limited to, retailers, the subsequent consumer purchaser of the product from a retailer or remote purchaser, to obligate Beurer in any way beyond the terms set forth herein. This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate in connection with any products or parts thereof which are repaired, replaced, altered, or modified, without the prior explicitly written consent of Beurer.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

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